


AC Cord Retainer

Assembly Instructions



Contents


AC cord retainer (1)

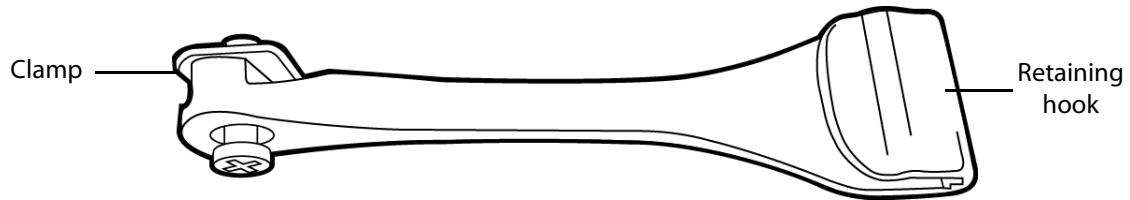

Attaching the AC cord retainer to the system stand


STEP 1

Insert the clamp into the stand base.

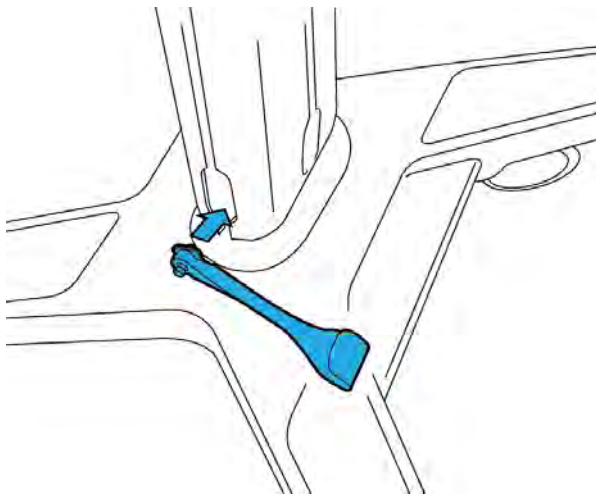


STEP 2

Tighten the clamp with a screwdriver

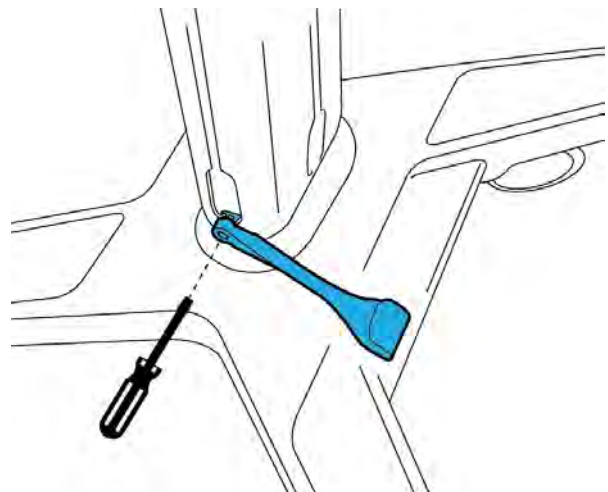


STEP 3

Insert the cord into the retaining hook.

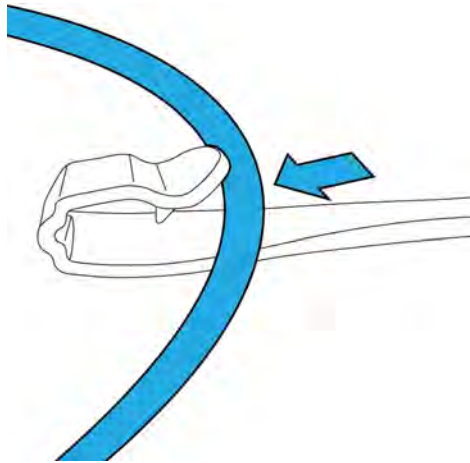

STEP 4

Plug the cord into the stand base, and then arrange the cord as shown.

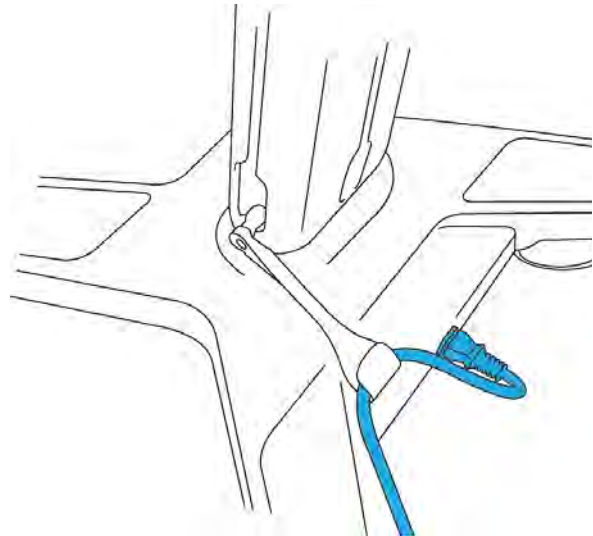

Cleaning, storage, and disposal

- For cleaning instructions, see the system stand user guide.
- Storage requirements: Temperature: -35 °C to 65 °C; Humidity: 15% to 95%; Atmospheric pressure: 500 hPa to 1060 hPa.
- To dispose of the AC cord retainer, contact FUJIFILM SonoSite, or your authorized representative.

SonoSite and the SonoSite logo are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

Publication date: November 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

Manufacturer
FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or
1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative
FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor
FUJIFILM SonoSite
Australasia Pty Ltd
114 Old Pittwater Road
Brookvale, NSW, 2100
Australia

P21632-02

