Bar Code Scanner

User Guide

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or
1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

SonoSite, SonoSite SII, X-Porte, Edge, M-Turbo, MicroMaxx, NanoMaxx, S Series, and the SonoSite logo are registered and unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

Part Number: P07766-08

Publication Date: November 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All Rights reserved.

Bar Code Scanner User Guide

Introduction	1
Additional documentation	1
Document conventions	2
Getting Help	2
Using the bar code scanner to enter patient information	3
Safety	3
Cleaning	3
Specifications	4

Introduction

This *Bar Code Scanner User Guide* supplements the manufacturer's documentation included with the bar code scanner by providing information about using the bar code scanner with the following ultrasound systems:

Table 1: Bar code scanners used with SonoSite ultrasound systems

Bar code scanner	Ultrasound system							
	X-Porte	NanoMaxx	Edge	SonoSite Edge II	S Series	SonoSite SII	M-Turbo	MicroMaxx
JADAK flexpoint HS-1M*	✓	✓	✓	✓		✓		
Symbol LS2208*		✓			✓		✓	
Symbol DS6707**								✓

* USB bar code scanner
** Serial bar code scanner

Additional documentation

For information from the manufacturer, see one of these documents:

- ▶ *JADAK flexpoint HS-1M Quick Start Guide*, included with the bar code scanner
- ▶ *Symbol LS 2208 Product Reference Guide*, included with the bar code scanner and available from Motorola at www.symbol.com/manuals

- ▶ *DS6707 Digital Imager Scanner Product Reference Guide*, available from Motorola at www.symbol.com/manuals

To set up the bar code scanner on the ultrasound system, see one of these documents:

- ▶ USB: *USB Bar Code Scanner Installation Instructions* from SonoSite
- ▶ Serial: *Ultrasound system user guide*

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the user guide.

Getting Help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451

Printed in the U.S.

Using the bar code scanner to enter patient information

You can use the bar code scanner to automatically enter patient information by scanning the patient's bar code. When you scan the bar code, the patient's ID and other information associated with the ID, if any, populate on the patient information form, saving time and preventing errors.

On certain systems, scanning the patient ID bar code can also query the worklist for patient information. To enable this functionality, do one of the following:

- ▶ (X-Porte) Make sure that **Bar Code Auto Query** is selected on the User Profile setup page and that a worklist server is configured and connected. For more information, see the ultrasound system Help.
- ▶ (SonoSite Edge II, SonoSite SII, and Edge) Make sure that **Bar Code Auto Lookup** is selected on the Connectivity setup page and that a worklist server is configured and connected. For more information, see the document *Sending and Receiving DICOM Data on SonoSite Systems*.

To use the bar code scanner to enter patient information

- 1 Tap **Patient**.
- 2 Ensure that your cursor is in the **Patient ID** text box.
- 3 Scan the patient's bar code.

Note

If your cursor is not in the Patient ID text box and the Bar Code Auto Query/Lookup on the User Profile or the Connectivity setup page is not selected, the scan of the patient's bar code appears in whatever text box is selected.

- 4 Ensure that the entered patient ID is correct. If it is incorrect, you can enter the patient ID manually.

Safety

WARNINGS

- ▶ To avoid damage to the eye, do not stare into the beam. The bar code scanner is a Class 2 laser product.
- ▶ Do not allow patient contact with the bar code scanner.

Cleaning

The exterior surface and cable of the bar code scanner can be cleaned using the recommended cleaning product.

To clean the JADAK bar code scanner

- ❖ Follow the cleaning instructions in *JADAK flexpoint HS-1M Quick Start Guide*.

To clean a Symbol bar code scanner

Caution

To avoid damaging the bar code scanner, do not immerse the bar code scanner in the cleaning or disinfecting solutions.

- 1 Disconnect the scanners.
 - ▶ (USB) Disconnect the scanner from the USB port.
 - ▶ (Serial) Disconnect the AC power from the bar code scanner power supply.
- 2 Clean the surface and cable using a clean towel and one of the following cleaning products. Apply the cleaning product to the cloth rather than the surface.
 - ▶ Sani-Cloth HB
 - ▶ Sani-Cloth Plus
 - ▶ Super Sani-Cloth
 - ▶ Isopropyl Alcohol Wipes (70%)
 - ▶ Caviwipes
 - ▶ Betadine
 - ▶ 409
 - ▶ Windex Blue
 - ▶ Hydrogen Peroxide (3%)
 - ▶ Clorox Bleach
 - ▶ Gentle Dish Soap and Water
- 3 Air dry or towel dry with a clean cloth.

For cleaning the lens on the Symbol bar code scanner, refer to *Symbol LS 2208 Product Reference Guide* or *DS6707 Digital Imager Scanner Product Reference Guide*.

Specifications

The ultrasound system supports the bar code scanning formats Code 39 and Code 128.

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