M-Turbo Ultrasound System User Guide Errata P20052-03

The following content was not included in the system user guide; it will be added to the next update.

Guidance and manufacturer's declaration

Table 1: Manufacturer's Declaration - Electromagnetic Environment

Emissions Test	Compliance	Electromagnetic Environment
RF emissions CISPR 11	Group 1	The SonoSite ultrasound system uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The SonoSite ultrasound system is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network which supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Recommended disinfectants

Table 2: Disinfectant Compatibility with System and Transducers

Disinfection and Cleaning Solutions	L38xi
Nanosonics Trophon EPR*#	✓

✓ = Acceptable for use.
* = For information and guidance on disinfection effectiveness and appropriate clinical use, see the manufacturer's user manual.
= Not approved for use in Canada.

Table 3: Disinfectant Compatibility with System and Transducers

Disinfection and Cleaning Solutions	L38xi
Accel TB Wipes	⊘

Table 3: Disinfectant Compatibility with System and Transducers

Disinfection and Cleaning Solutions	L38xi
Accel Intervention Wipes	⊗
Aikem A-33	✓
Alcohol, Ethanol	✓
Alcohol, Isopropyl	⊗
Anios Wipes	⊗
Aniosyme DD1-5%	✓
Anioxide 1000	⊗
Asepti-HB	✓
Asepti-Steryl	⊗
Asepti-Wipe II	⊗
Baxedin Pre-Op	✓
Carpe Diem TB Wipes	⊗
Caviwipes	✓
Cidex	✓
Cidex OPA	✓
Cidex Plus	✓
Clinell Universal Sanitising Wipes	✓
Clorox Germicidal Wipes	✓
Clorox Hydrogen Peroxide Wipes	⊗
Coverage Germicidal Surface Wipes	✓
Coverage Plus Germicidal Surface Wipes	✓
Coverage Spray	✓
Coverage Spray TB	✓
Dynacide PA	✓
Echo Clean Lingettes	⊗
End-Bac II	⊗
Endosporine	✓

Table 3: Disinfectant Compatibility with System and Transducers

Disinfection and Cleaning Solutions	L38xi
Envirocide	✓
Gluco Chlor Wipes	✓
Hydrogen Peroxide 3%	✓
Incidin Plus 1%	✓
Incidin Plus 3%	✓
Madacide 1	✓
Medzyme	✓
Metricide OPA Plus	✓
Metrisponge	✓
Mikrobac Forte 2%	✓
Mikrobac Tissues	✓
NU - Cidex	⊘
Oxivir TB Wipes	⊘
Percept RTU Wipes	✓
Protex Spray	✓
Protex Wipes	✓
Rely+On Perasafe	⊘
Revital-Ox Resert HLD	⊘
Sanicloth AF	⊘
Sanicloth AF3	✓
Sanicloth Bleach Wipes	✓
Sanicloth HB	✓
Sanicloth Plus	⊘
Sporicidin	⊘
Staphene	⊘
Steranios 2%	✓
Steranios 20%	⊘
Super Sanicloth	⊘
Trigene Advance Wipes	✓
Tristel Solo	✓

Table 3: Disinfectant Compatibility with System and Transducers

Disinfection and Cleaning Solutions	L38xi
Tristel Wipes	⊗
T-Spray	✓
T-Spray II	✓
Virex II 256	✓
Virex TB	✓
Virox 5 RTU Wipes	✓
Wet Task Wipes	✓
Wipes Plus Disinfecting Wipes	✓

✓ = Acceptable
⊗ = Unacceptable. Do not use

Chapter 8: Specifications

The following content has been corrected.

Electromechanical safety standards

IEC 60601-1:2005, Medical electrical equipment - Part 1: General requirements for basic safety and essential performance (3rd Edition including corrigendum 1 and 2).

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消毒和清洁溶液	L38xi
醇类, 异丙醇	⊗
Anios 湿巾	⊗
Aniosyme DD1-5%	✓
Anioxide 1000	⊗
Asepti-HB	✓
Asepti-Steryl	⊗
Asepti-Wipe II	⊗
Baxedin Pre-Op	✓
Carpe Diem TB 湿巾	⊗
Caviwipes	✓
Cidex	✓
Cidex OPA	✓
Cidex Plus	✓
Clinell Universal 卫生湿巾	✓
Clorox 杀菌湿巾	✓
Clorox 过氧化氢湿巾	⊗
Coverage 杀菌表面湿巾	✓
Coverage Plus 杀菌表面湿巾	✓
Coverage 喷雾	✓
Coverage 喷雾 TB	✓
Dynacide PA	✓
Echo Clean 湿巾	⊗
End-Bac II	⊗
Endosporine	✓
Envirocide	✓
Gluco Chlor 湿巾	✓
过氧化氢 3%	✓
Incidin Plus 1%	✓
Incidin Plus 3%	✓

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Australia Sponsor
FUJIFILM SonoSite
Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

02/2018
Copyright © 2018
FUJIFILM SonoSite, Inc.
All rights reserved.

Caution | United States federal law restricts this device to sale by or on the order of a physician.

M-Turbo, SonoSite and the SonoSite logo are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a trademark of FUJIFILM Corporation.

P20052-03

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FUJIFILM SonoSite B.V.
1114 AB Amsterdam,