


Transducer Port Cover Installation Instructions

FUJIFILM
Value from Innovation

SonoSite

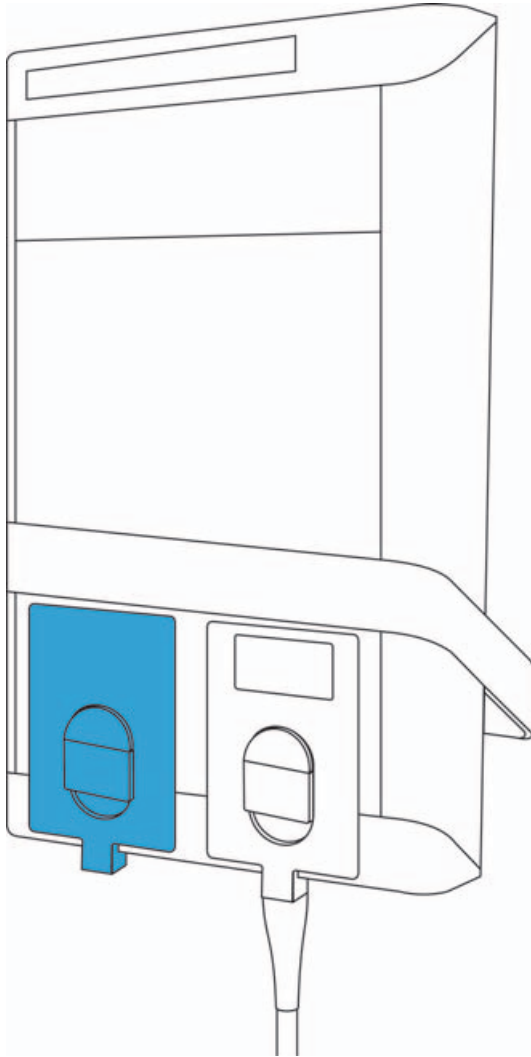


Transducer Port Cover installed on
SonoSite SII system

**Can be used with the following systems
and accessories:**

- ▶ SonoSite M-Turbo
- ▶ SonoSite S Series
- ▶ SonoSite SII
- ▶ SonoSite Edge
- ▶ SonoSite Edge II
- ▶ SonoSite MicroMaxx
- ▶ Triple Transducer Connect (TTC)

Contents


Transducer Port Cover

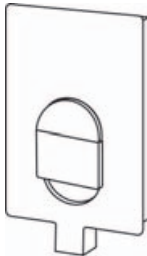

Attaching the Transducer Port Cover to the system


STEP 1

On the Transducer Port Cover, lift the latch and rotate it clockwise.

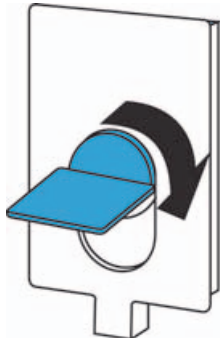


STEP 2

Insert the Transducer Port Cover into an empty transducer port on the back or underside of the system.

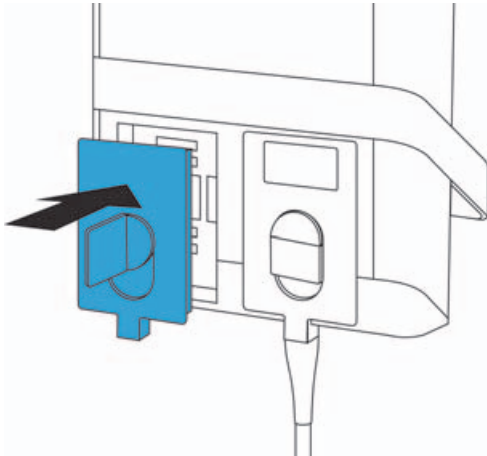


STEP 3

Turn the latch counter-clockwise.

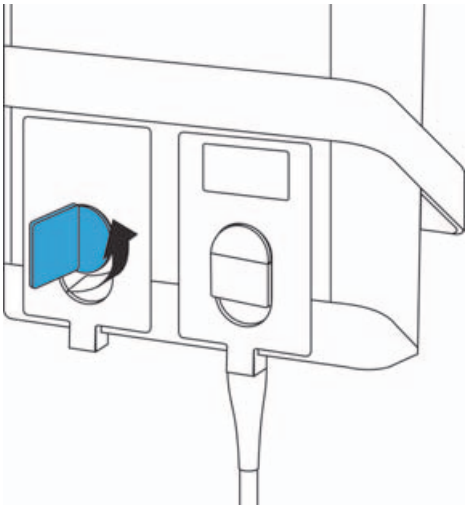

STEP 4

Press the latch down, securing the Transducer Port Cover to the system.

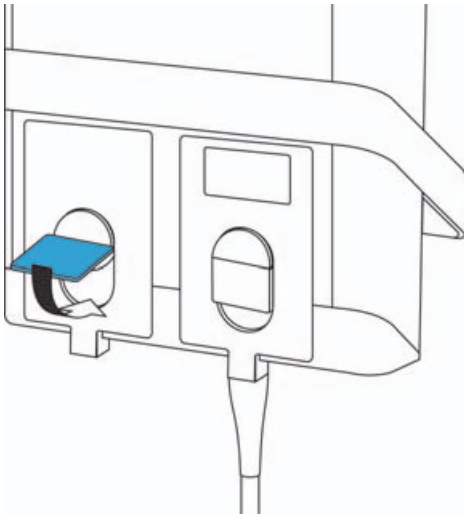

For cleaning, storage, and disposal

- ▶ For cleaning instructions, see the system user guide.
- ▶ Storage requirements: Temperature: -35°C (-31°F) to 65°C (149°F); Relative humidity: 15% to 95%; Atmospheric pressure: 500 hPa to 1060 hPa (.5 to 1.05 atmospheres).
- ▶ To dispose of the Transducer Port Cover, contact FUJIFILM SonoSite, or your authorized representative.

Manufacturer

FUJIFILM SonoSite, Inc.

21919 30th Drive SE

Bothell, WA 98021 USA

T: 1-888-482-9449 or 1-425-951-1200

F: 1-425-951-1201

EC Authorized Representative

Emergo Europe

Molenstraat 15

2513 BH, The Hague

The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd

114 Old Pittwater Road

BROOKVALE, NSW, 2100

Australia

Caution | Federal (United States) law restricts this device to sale by or on the order of a physician.

SonoSite Edge, SonoSite S Series, SonoSite SII, SonoSite M-Turbo, SonoSite MicroMaxx, SonoSite, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.



Publication Date: November 2016

Copyright © 2016 FUJIFILM SonoSite, Inc. All rights reserved.

P21461-01

