



SonoSite X-Porte AC Cord Retainer Installation Instructions

FUJIFILM
Value from Innovation

SonoSite

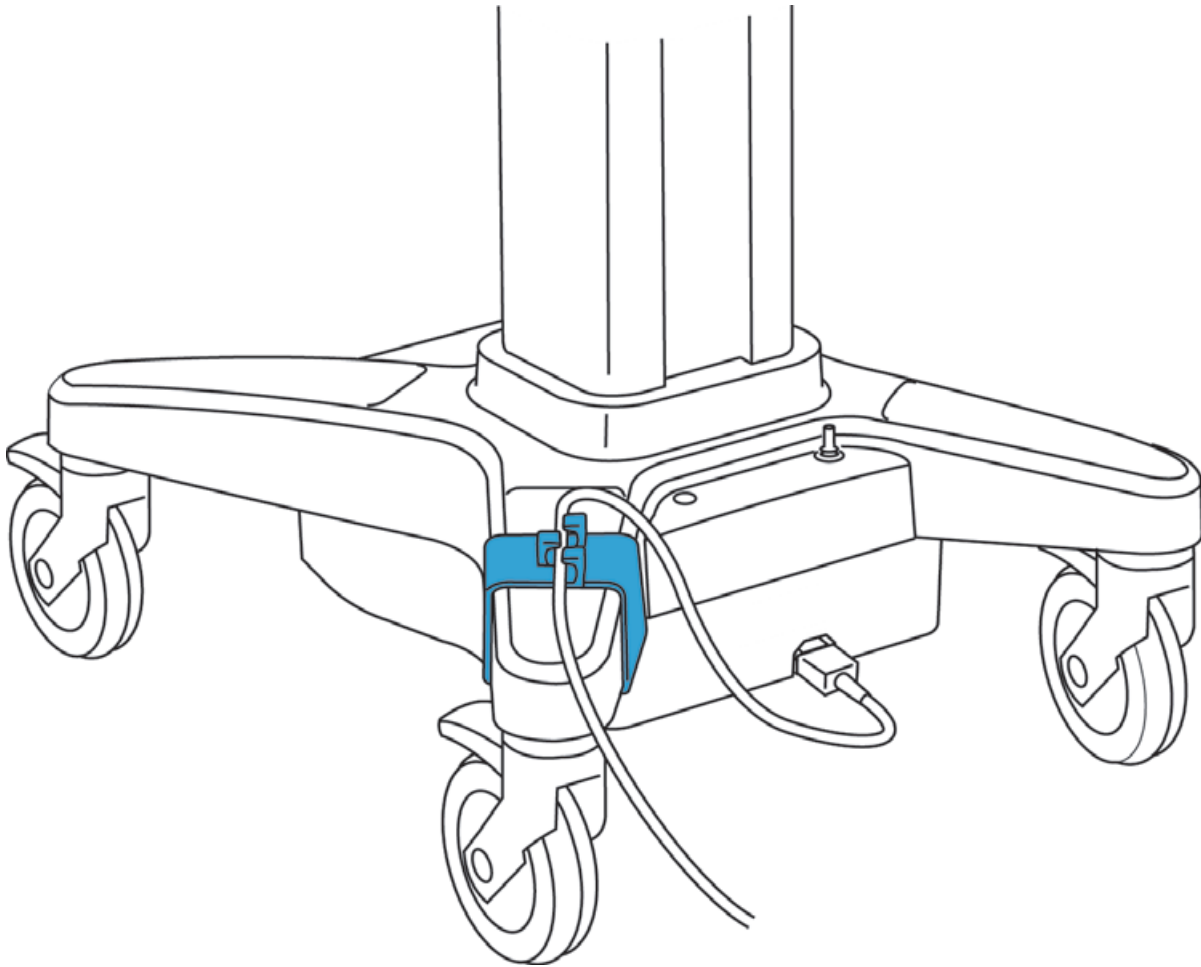

Contents

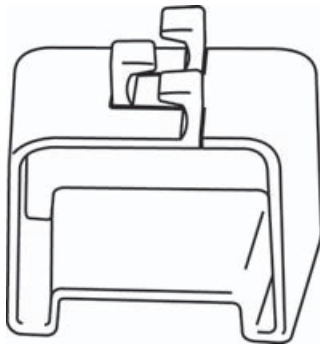


AC Cord Retainer (1)

Preparing the stand

STEP 1


Press down the lever on two or more wheels.

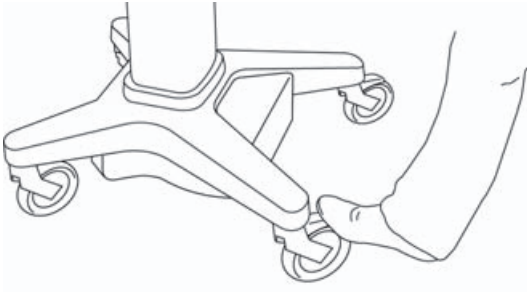

Attaching the AC cord retainer


STEP 2

Find the right-rear leg of the stand.

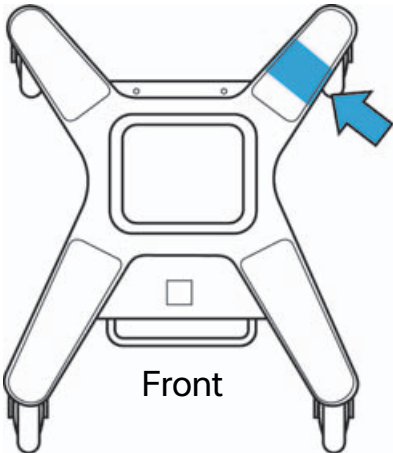


STEP 3

Spread the AC Cord Retainer apart.

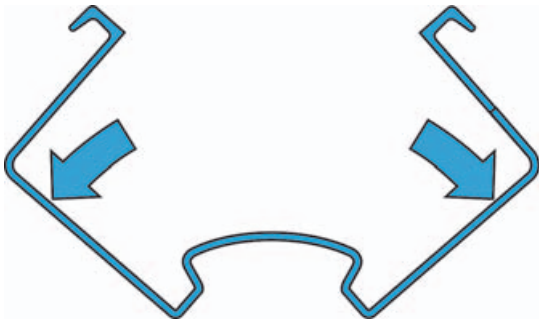


STEP 4

Wrap the AC Cord Retainer around the right-rear leg of the stand.

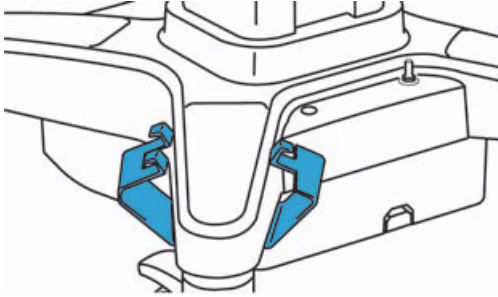


STEP 5

Close the AC Cord Retainer around the stand leg.

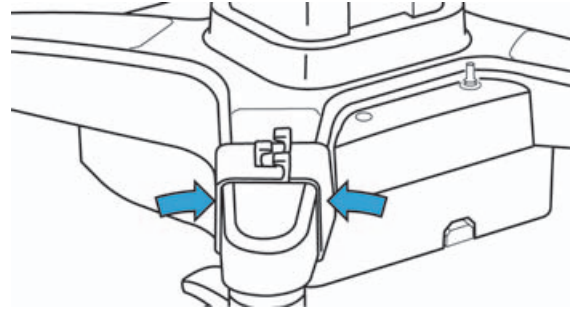


STEP 6

Pass the AC power cord through the hooks in the AC Cord Retainer. The arrangement of the cord may differ, depending on your configuration. See the examples below.

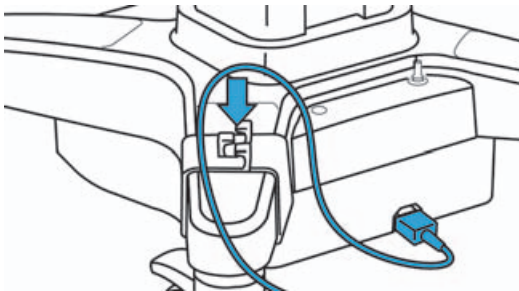


EXAMPLE A

AC power cord attached to the stand base.

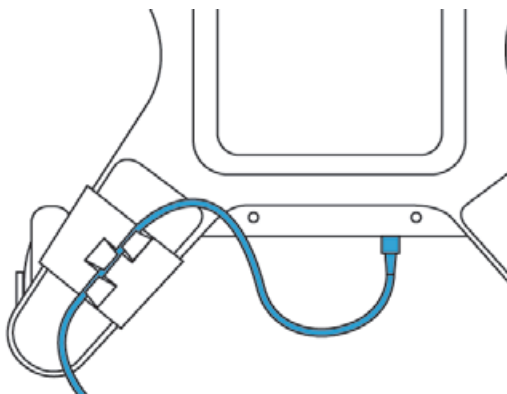

EXAMPLE B

AC power cord attached to a PowerPark stand module.

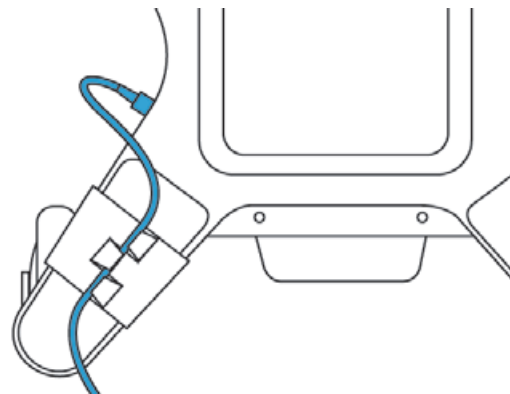

For cleaning, storage, and disposal

- ▶ For cleaning instructions, see the system user guide.
- ▶ Storage requirements: Temperature: -35°C to 65°C; Humidity: 15% to 95%; Atmospheric pressure: 500 hPa to 1060 hPa.
- ▶ To dispose of the AC cord retainer, contact FUJIFILM SonoSite, or your authorized representative.

Caution

Federal (United States) law restricts this device to sale by or on the order of a physician.

X-Porte, SonoSite, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Publication Date: November 2016

Copyright © 2016 FUJIFILM SonoSite, Inc. All rights reserved.

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

Australia Sponsor

FUJIFILM SonoSite
Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia



P22734-01

