



SonoSite X-Porte Stand: Mobile Printer Installation Instructions

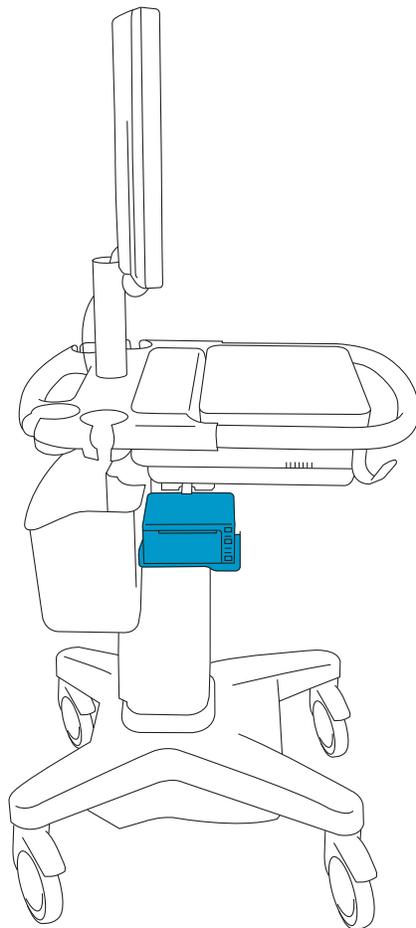


*Mobile printer installed on
the SonoSite X-Porte stand*

Unpacking the box


Printer box

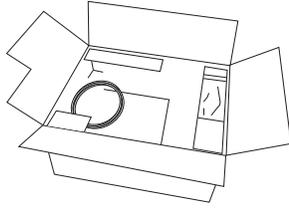


Sony USB cable
(do not use)

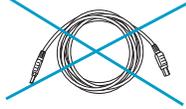

Printer (1)

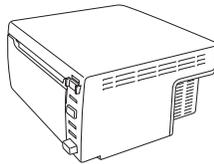


Printer paper (1)

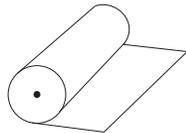


Sony user documentation



Printer bracket box

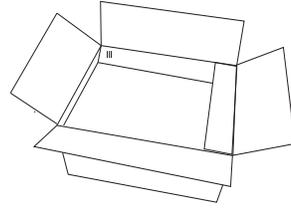

Printer bracket with attached
printer and dock cables (1)

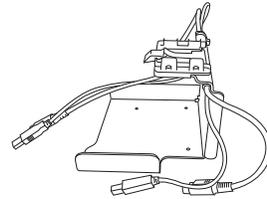

Phillips-head screw (4)



3 mm Allen wrench (1)




#1 Phillips screwdriver (1)



Assembling the printer


STEP 1

Remove the screws that come with the printer.

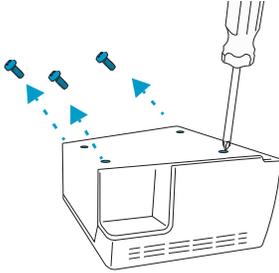


STEP 2

Slide the bracket onto the printer, aligning the four holes.

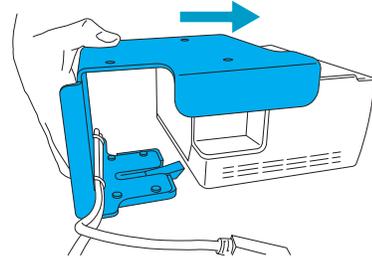


STEP 3

Screw in the four Phillips head screws.

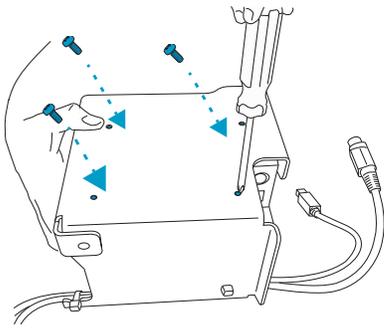

STEP 4


Plug the printer USB and power cables that are attached to the bracket into the printer.

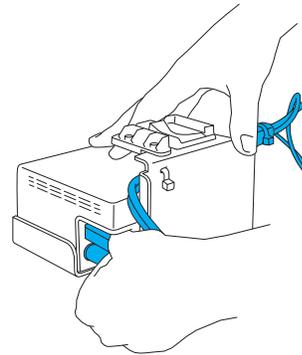

Installing the printer on the stand


STEP 5

Turn off the system, and unplug it from the wall. Remove all USB accessories.

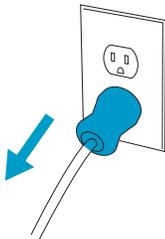


STEP 6

Loosen the basket with the Allen wrench, and lower all the way down.

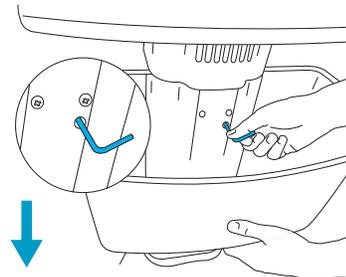


STEP 7

Snap the printer into place below the stand head.

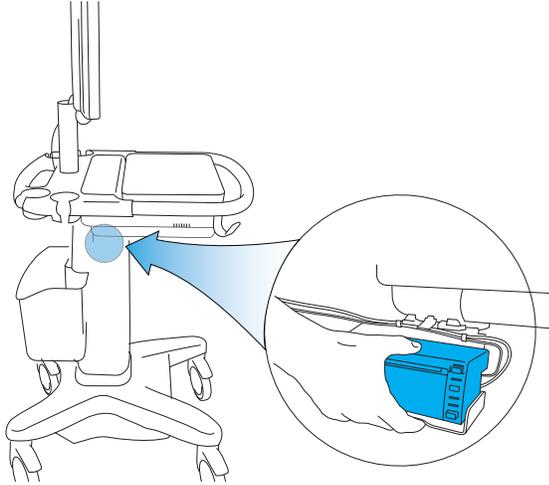


STEP 8

Disconnect the triple transducer connector (TTC) from the engine by pulling the latch up and rotating it clockwise.

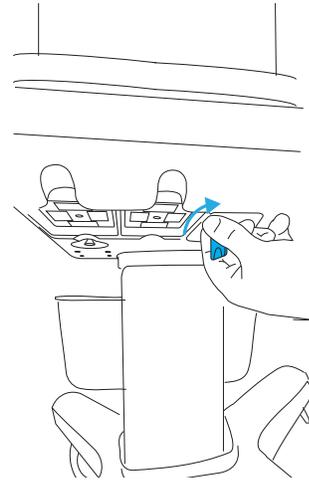


STEP 9

To loosen the engine from the stand, raise the control panel then release the two latches on either side of the stand head.

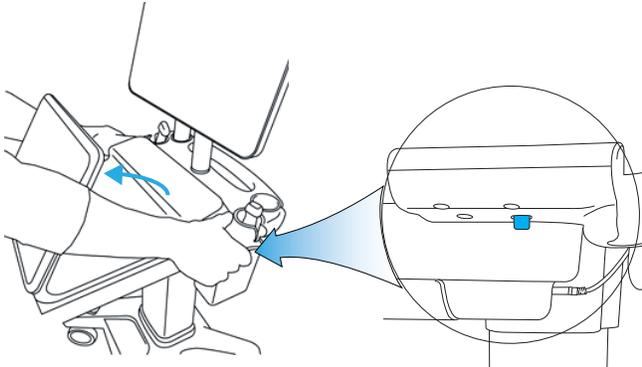


STEP 10

Remove the engine by lifting the back end out first.

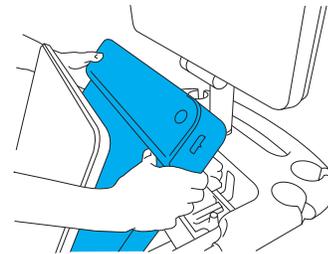


STEP 11

Remove the strain relief that holds the dock cables in place.

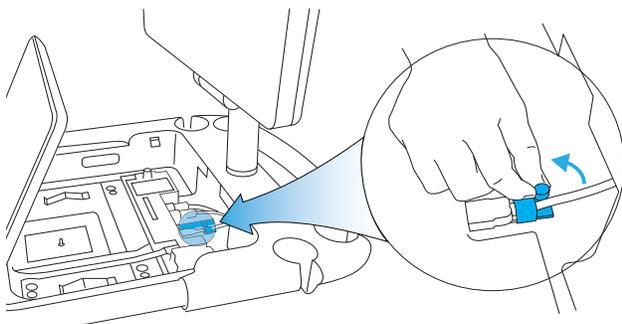


STEP 12

Push the power and USB cables attached to the printer bracket into the back of the stand head.

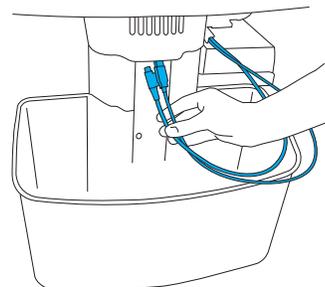


STEP 13

Plug the cables into the dock.

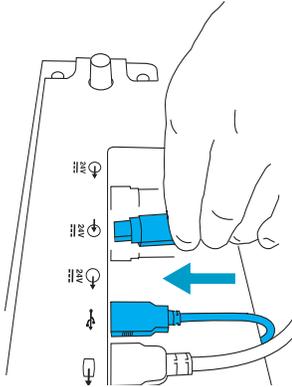

STEP 14

Reinstall the strain relief.

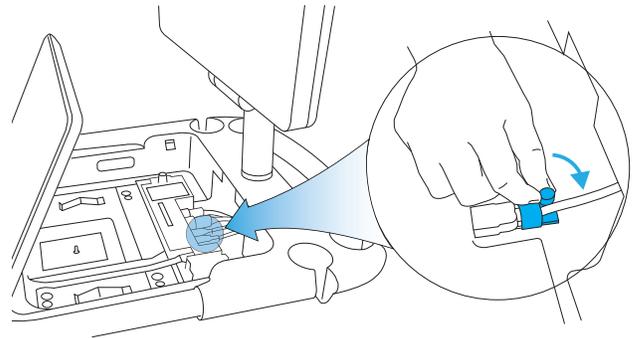


STEP 15

Reinstall the engine by inserting the front end first. Push down on both sides of the back end until the latches click into place.




STEP 16

Reconnect the TTC to the engine.
Raise and tighten the basket.

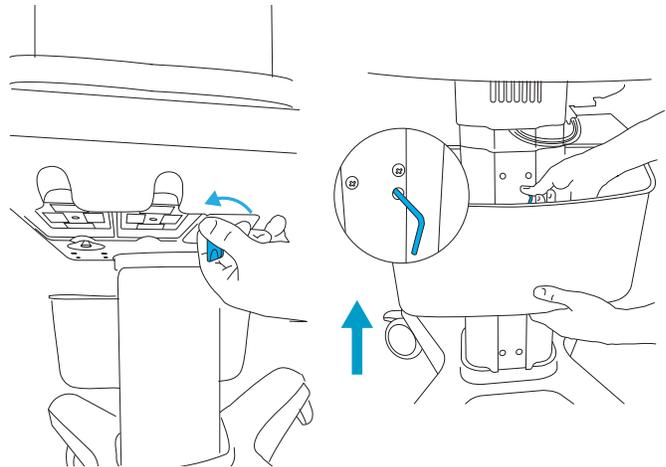


STEP 17

Plug the system AC power cord into the wall, and turn the system back on.
Reinstall your USB devices.

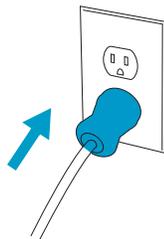

STEP 18


Press the Power button on the printer, then adjust the printer settings under the system settings.

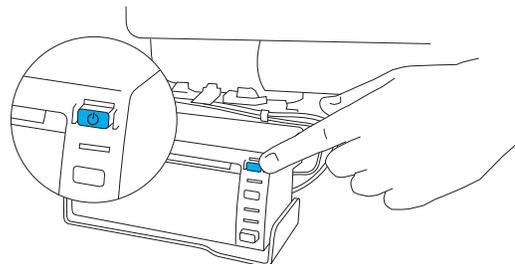

Uninstalling the printer


STEP 1

Turn off the system, and unplug it from the wall. Remove all USB accessories.

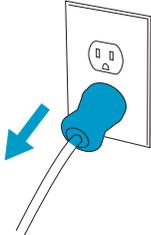


STEP 2

Loosen and lower the basket.

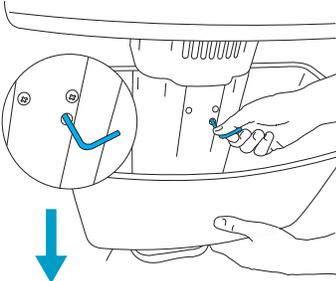


STEP 3

Disconnect the TTC from the engine by pulling the latch up and rotating it clockwise.

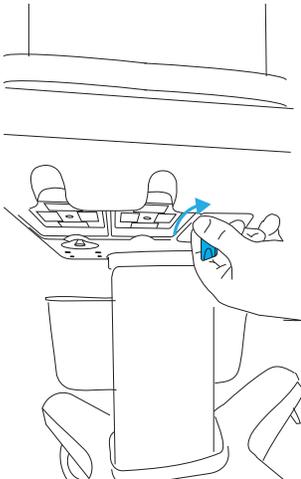


STEP 4

To loosen the engine from the stand, raise the control panel then release the two latches on either side of the stand head.

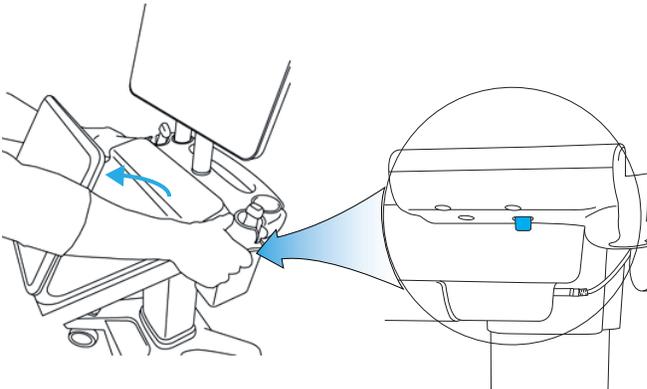


STEP 5

Remove the engine by lifting the back end out first.

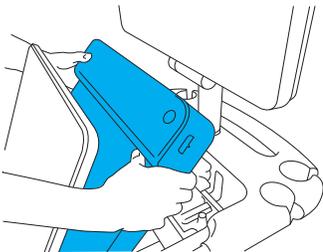


STEP 6

Remove the strain relief.

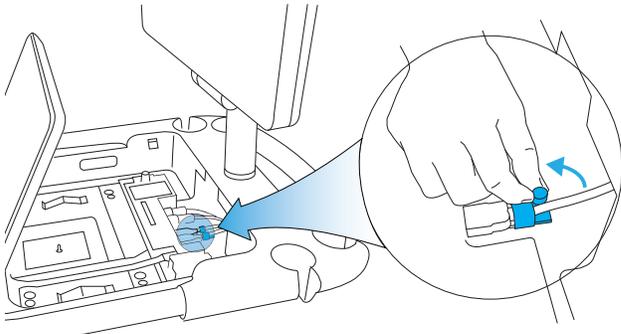


STEP 7

Disconnect the printer cables. To disengage the power cord, first slide the jacket back from the end.

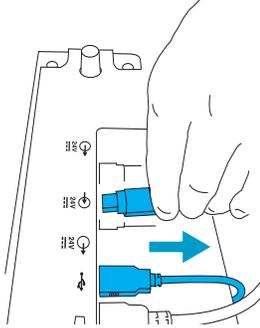


STEP 8

Press the top part of the printer bracket down, and slide the printer out.

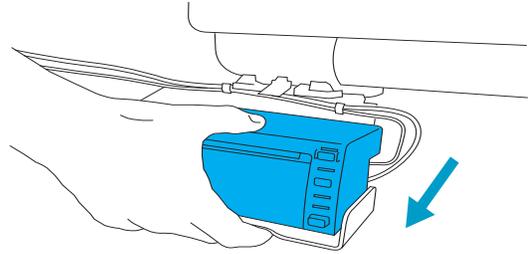


STEP 9

Reinstall the strain relief and engine.

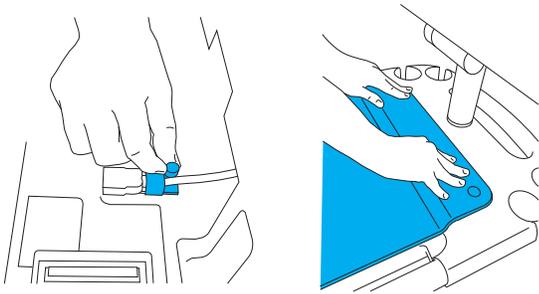


STEP 10

Reconnect the TTC to the engine.
Raise and tighten the basket.

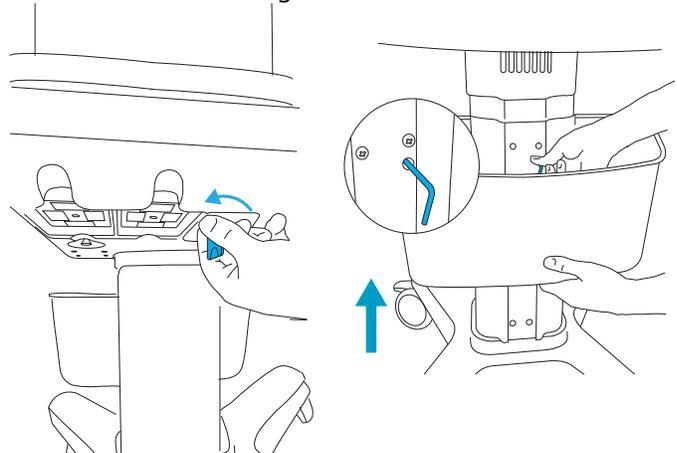

STEP 11

Reconnect the TTC to the engine, and plug the system AC power cord into the wall.

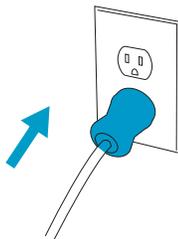

Caution

Federal (United States) law restricts this device to sale by or on the order of a physician.

X-Porte, SonoSite, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

Patents: US 8,398,408; US 7,591,786; US D625014; and US D625015.

Published January 2017

Copyright © 2016 FUJIFILM SonoSite, Inc. All rights reserved.

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

Australia Sponsor

FUJIFILM SonoSite
Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia



P22657-01

