

Disinfectants for Retired FUJIFILM SonoSite Products



Table of Contents

Introduction	1
Retired Ultrasound Systems and Docks	2
Standard Retired Transducers, A-H	8
Standard Retired Transducers, I-Z	12
Retired Specialty Transducers	17
Retired Accessories	21
Cleaning Transducers	26

Introduction

FUJIFILM SonoSite has tested disinfectants and disinfectant devices for use with its retired systems, transducers, and accessories. Except for TEE and surgical transducers, the results are in the tables below. See the ultrasound system user guide for cleaning and disinfecting instructions and for additional warnings and cautions.

- For cleaners and disinfectants compatible with TEE and surgical transducers, see the *TEE Care Card* and *Surgical Transducer Guide*.
- For information about current FUJIFILM SonoSite products, refer to the *Cleaning and Disinfecting FUJIFILM SonoSite Products User Guide*.

WARNING: Disinfectants listed are recommended by FUJIFILM SonoSite for safe use with FUJIFILM SonoSite products.
Refer to instructions on the disinfectant label for guidance on disinfection effectiveness and appropriate clinical use.

This document does not have the following regulatory information for disinfectants:

- EPA registration
- FDA 510(k) clearance (liquid sterilant, high level disinfectant)
- CE approval

Before using a disinfectant, confirm that its regulatory status is appropriate for your jurisdiction and use. Verify expiration dates on chemicals.

When disposing of chemicals, follow manufacturer recommendations and EPA regulations.

Retired Ultrasound Systems and Docks

You can disinfect the exterior surface of the ultrasound system using a recommended disinfectant.

Table 1: Disinfectant Compatibility with Retired FUJIFILM SonoSite Ultrasound Systems and Docks

Disinfectant	MicroMaxx (system and dock)	TITAN (system and dock)	SonoSite 180PLUS	iLook15 iLook25
AbcoCide 14				✓
Airkem A-33	✓	✓	✓	✓
Alcohol, Ethanol				
Alkacide			✓	
Alkazyme			✓	
Anios Wipes	✓			✓
Anioxyde 1000	✓		✓	
Aniosyme DD1 (5%)				
Aquatabs (1000 PPM)			✓	
Ascend			✓	✓
Asepti-HB				
Asepti-Steryl				
Asepti-Wipes		✓		
Asepti-Wipe II				
Bacillocid rasant			✓	✓
Banicide				✓

Table 1: Disinfectant Compatibility with Retired FUJIFILM SonoSite Ultrasound Systems and Docks (continued)

Disinfectant	MicroMaxx (system and dock)	TITAN (system and dock)	SonoSite 180PLUS	iLook15 iLook25
Bacoban WB			✓	
Baxedin Pre-Op	✓	✓		
BioText Micrylium			✓	
CaviWipes	✓	✓	✓	
Cidalkan	✓	✓		
Cidalkan Lingettes	✓	✓	✓	
Cidex	✓	✓		
Cidex 14			✓	✓
Cidex OPA		✓		✓
Cidex Plus	✓	✓	✓	✓
Cleanisept Wipe				✓
Clinell Universal Sanitising Wipes	✓	✓		
Clorox Germicidal Wipes		✓		
Control III	✓	✓		
Coverage Germicidal Surface Wipes				
Coverage Plus Germicidal Surface Wipes		✓		
Coverage Spray				
DentaSept			✓	

Table 1: Disinfectant Compatibility with Retired FUJIFILM SonoSite Ultrasound Systems and Docks (continued)

Disinfectant	MicroMaxx (system and dock)	TITAN (system and dock)	SonoSite 180PLUS	iLook15 iLook25
Discide Ultra Wipes				
DisOPA				✓
Dispatch Wipes	✓			
Dynacide PA	✓	✓	✓	✓
Echo Clean Lingettes				
End-Bac II	✓	✓		
Endosporine	✓			✓
EndoZime AW Plus			✓	✓
Envirocide				
Enzol			✓	✓
Expose II			✓	
Gluco Chlor Wipes	✓			
Hexanios			✓	✓
Hi Tor Plus			✓	
Hibiclens			✓	✓
Hydrogen Peroxide (3%)				
Incidin Plus (1%)	✓	✓	✓	
Incidin Plus (3%)	✓		✓	

Table 1: Disinfectant Compatibility with Retired FUJIFILM SonoSite Ultrasound Systems and Docks (continued)

Disinfectant	MicroMaxx (system and dock)	TITAN (system and dock)	SonoSite 180PLUS	iLook15 iLook25
Korsolex Extra			✓	✓
MadaCide 1				
Medzyme	✓	✓		
MetriCide 14			✓	✓
MetriCide 28			✓	✓
MetriCide OPA Plus		✓		✓
MetriZyme			✓	✓
Mikrobac forte			✓	✓
Mikrobac forte 2%				
Mikrobac Tissues				
Nu-Cidex	✓	✓		
Nuclean			✓	✓
PerCept RTU Wipes	✓	✓		
Rely+On PeraSafe		✓		✓
Salvanios pH 7	✓		✓	✓
Sani-Cloth AF				
Sani-Cloth HB				
Sani-Cloth Bleach Wipes	✓			

Table 1: Disinfectant Compatibility with Retired FUJIFILM SonoSite Ultrasound Systems and Docks (continued)

Disinfectant	MicroMaxx (system and dock)	TITAN (system and dock)	SonoSite 180PLUS	iLook15 iLook25
Sani-Cloth Plus	✓			✓
Sekusept aktiv		✓		
Simple Chlor				
Sporicidin			✓	
Steranios 2%	✓	✓		✓
Steranios 20%	✓	✓		✓
Super Sani-Cloth				
T-Spray	✓	✓		
T-Spray II	✓	✓	✓	✓
TASK 105			✓	✓
Trigene Advance Wipes	✓	✓		✓
Tristel Solo	✓	✓	✓	✓
Tristel Wipes	✓	✓		
Virex 256				
Virex II 256			✓	
Virex TB				
Virox 5 Wipes	✓	✓	✓	
Virufen			✓	✓

Table 1: Disinfectant Compatibility with Retired FUJIFILM SonoSite Ultrasound Systems and Docks (continued)

Disinfectant	MicroMaxx (system and dock)	TITAN (system and dock)	SonoSite 180PLUS	iLook15 iLook25
Wavicide -01				✓
Wavicide -06			✓	
Wet Task Wipes	✓	✓		
Wet Wipe Disinfection	✓	✓	✓	✓
Wex-Cide		✓	✓	✓
Wipes Plus Disinfecting Wipes	✓	✓		
Wood Wyant Ultra Wipes				
✓ - Acceptable for use				

Standard Retired Transducers, A-H

You can disinfect the transducer, scanning surface, and cable using a recommended disinfectant or device. Clean the transducer prior to disinfecting to remove any particulate matter or bodily fluids. See [“To clean a transducer”](#) on page 26.

The retired transducers listed below correspond to those listed in the table [“Retired Ultrasound Systems and Docks”](#) on page 2.

Table 2: Disinfectant-Device Compatibility with Retired FUJIFILM SonoSite Standard Transducers (A-H)

Device	C11 C11e	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e	HFL38
Nanosonics Trophon EPR*	✓		✓		✓	✓

✓ - Acceptable for use

1- Part number: P01960-xx, P02645-02, P02645-03, or P03361-10

2 - Part number: P00798-03

* For information and guidance on disinfection effectiveness and appropriate clinical use, see the manufacturer’s user manual.

Table 3: Disinfectant Compatibility with Retired FUJIFILM SonoSite Standard Transducers (A-H)

Disinfectant	C11 C11e	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e	D2	HFL38
Airkem A-33	✓	✓	✓	✓	✓	✓	✓
Alcohol, Ethanol							
Anios Wipes	✓	✓	✓	✓	✓	✓	✓
Aniosyme DD1 (5%)	✓	✓	✓	✓	✓		✓
Anioxyde 1000						✓	
Asepti-HB							
Asepti-Wipe II	✓	✓	✓	✓	✓		✓
Bacoban	✓	✓	✓	✓	✓	✓	✓

Table 3: Disinfectant Compatibility with Retired FUJIFILM SonoSite Standard Transducers (A-H) (continued)

Disinfectant	C11 C11e	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e	D2	HFL38
Bacoban WB						✓	
Baxedin Pre-Op	✓	✓	✓	✓	✓		✓
CaviWipes	✓	✓	✓	✓	✓		✓
Cidalkan	✓	✓	✓	✓	✓		✓
Cidalkan Lingettes	✓	✓	✓	✓	✓		✓
Cidex	✓	✓	✓	✓	✓	✓	✓
Cidex OPA	✓	✓	✓	✓	✓	✓	✓
Cidex Plus	✓	✓	✓	✓	✓	✓	✓
Cleanisept Wipes	✓	✓	✓	✓	✓	✓	✓
Clinell Universal Sanitising Wipes	✓	✓	✓	✓	✓	✓	✓
Clorox Wipes				✓			
Clorox Germicidal Wipes	✓	✓	✓	✓	✓		✓
Control III	✓	✓	✓	✓	✓		
Coverage Germicidal Surface Wipes	✓	✓	✓	✓	✓		✓
Coverage Plus Germicidal Surface Wipes	✓	✓	✓	✓	✓		✓
Coverage Spray							
DisOPA						✓	
Dispatch	✓	✓		✓			
Dispatch Wipes			✓		✓	✓	✓
Dynacide PA	✓	✓	✓	✓	✓	✓	✓

Table 3: Disinfectant Compatibility with Retired FUJIFILM SonoSite Standard Transducers (A-H) (continued)

Disinfectant	C11 C11e	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e	D2	HFL38
Echo Clean Lingettes	✓	✓	✓	✓	✓		✓
End-Bac II	✓	✓	✓	✓	✓		✓
Endosporine	✓	✓	✓	✓	✓	✓	✓
Envirocide							
Gluco Chlor Wipes	✓	✓	✓	✓	✓		✓
Hydrogen Peroxide (3%)	✓	✓	✓	✓	✓	✓	
Incidin Plus (1%)	✓	✓	✓	✓	✓	✓	✓
Incidin Plus (3%)	✓	✓	✓	✓	✓	✓	✓
Korsolex Extra	✓	✓	✓	✓	✓		✓
Madacide 1	✓	✓	✓	✓	✓		✓
Medzyme	✓	✓	✓	✓	✓	✓	✓
Metricide OPA Plus	✓	✓	✓	✓	✓	✓	✓
MetriSponge	✓	✓	✓	✓	✓	✓	✓
Nu-Cidex						✓	
PerCept RTU Wipes	✓	✓	✓	✓	✓	✓	✓
Rely+On PeraSafe						✓	
Sani-Cloth Bleach Wipes	✓	✓	✓	✓	✓	✓	✓
Sani-Cloth HB	✓	✓	✓	✓	✓		
Sani-Cloth Plus	✓	✓	✓	✓	✓		
Sekusept	✓	✓	✓	✓	✓	✓	✓
Staphene	✓	✓	✓	✓	✓		✓

Table 3: Disinfectant Compatibility with Retired FUJIFILM SonoSite Standard Transducers (A-H) (continued)

Disinfectant	C11 C11e	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e	D2	HFL38
Steranios (2%)	✓	✓	✓	✓	✓	✓	✓
Steranios 20%						✓	
T-Spray			✓		✓		
T-Spray II	✓	✓	✓	✓	✓	✓	✓
Trigene Advance Wipes	✓	✓	✓	✓	✓	✓	✓
Tristel Duo						✓	
Tristel Solo	✓	✓	✓	✓	✓	✓	✓
Tristel Wipes						✓	
Virex 256							
Virex TB							
Virox 5 Wipes	✓	✓	✓	✓	✓	✓	✓
Virufen						✓	
Wet Task Wipes	✓	✓	✓	✓	✓	✓	✓
Wet Wipe Disinfection	✓	✓	✓	✓	✓	✓	✓
Wex-Cide	✓	✓	✓	✓	✓		
Wipes Plus Disinfecting Wipes	✓	✓	✓	✓	✓	✓	✓

✓ - Acceptable for use

1 - Part number: P01960-xx, P02645-02, P02645-03, or P03361-10

2 - Part number: P00798-03

3 - Disinfectant has been tested by FUJIFILM SonoSite for disinfection effectiveness on FUJIFILM SonoSite transducers.

Standard Retired Transducers, I-Z

You can disinfect the transducer, scanning surface, and cable using a recommended disinfectant or device. Clean the transducer prior to disinfecting to remove any particulate matter or bodily fluids. See [“To clean a transducer”](#) on page 26.

Table 4: Disinfectant-Device Compatibility with Retired FUJIFILM SonoSite Standard Transducers (I-Z)

Device	L25 L25e	L38 L38e	L52 L52e Vet Only	P10	P17
Trophon EPR*	✓	✓		✓	✓

✓ - Acceptable for use

* For information and guidance on disinfection effectiveness and appropriate clinical use, see the manufacturer's user manual.

Table 5: Disinfectant Compatibility with Retired FUJIFILM SonoSite Standard Transducers (I-Z)

Disinfectant	L25 L25e	L38 L38e	L52 L52e Vet Only	P10	P17
Accel Intervention Wipes					
Airkem A-33	✓	✓	✓	✓	✓
Alcohol, Ethanol					
Anios Wipes	✓	✓	✓	✓	✓
Aniosyme DD1 (5%)	✓	✓	✓	✓	✓
Anioxyde 1000			✓		
Asepti-HB					
Asepti-Wipe II	✓	✓	✓	✓	✓
Bacoban	✓	✓	✓	✓	✓
Baxedin Pre-Op	✓	✓	✓	✓	✓

Table 5: Disinfectant Compatibility with Retired FUJIFILM SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e	L38 L38e	L52 L52e Vet Only	P10	P17
Carpe Diem TB Wipes					
Cavicide					
CaviWipes	✓	✓	✓	✓	✓
Cidalkan	✓	✓	✓	✓	✓
Cidalkan Lingettes	✓	✓	✓	✓	✓
Cidex	✓	✓	✓	✓	✓
Cidex OPA	✓	✓	✓	✓	✓
Cidex Plus	✓	✓	✓	✓	✓
Cleanisept Wipes	✓	✓	✓	✓	✓
Clinell Universal Sanitising Wipes	✓	✓		✓	✓
Clorox Germicidal Wipes	✓	✓	✓	✓	✓
Clorox Hydrogen Peroxide Wipes					
Control III	✓	✓	✓	✓	✓
Coverage Germicidal Surface Wipes	✓	✓	✓	✓	✓
Coverage Plus Germicidal Surface Wipes	✓	✓		✓	✓
Coverage Spray					
DisCide Ultra Disinfecting Towelettes					

Table 5: Disinfectant Compatibility with Retired FUJIFILM SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e	L38 L38e	L52 L52e Vet Only	P10	P17
Dispatch	✓				
Dispatch Wipes		✓	✓	✓	✓
Dynacide PA	✓	✓	✓	✓	✓
Echo Clean Lingettes	✓	✓	✓	✓	✓
End-Bac II	✓	✓	✓	✓	✓
Endosporine	✓	✓	✓	✓	✓
Envirocide					
Gluko Chlor Wipes	✓	✓	✓	✓	✓
Hydrogen Peroxide (3%)	✓	✓	✓	✓	✓
Incidin Plus (1%)	✓	✓	✓	✓	✓
Incidin Plus (3%)	✓	✓	✓	✓	✓
Korsolex Extra	✓	✓	✓	✓	✓
Madacide 1	✓	✓	✓	✓	✓
Medzyme	✓	✓	✓	✓	✓
Metricide OPA Plus	✓	✓	✓	✓	✓
MetriSponge	✓	✓	✓	✓	✓
MetriZyme					
Microbac forte 2%					
Mikrobac Tissues					
Oxivir TB Wipes					

Table 5: Disinfectant Compatibility with Retired FUJIFILM SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e	L38 L38e	L52 L52e Vet Only	P10	P17
PerCept RTU Wipes	✓	✓	✓	✓	✓
Protex Wipes					
Rely+On PeraSafe			✓		
Sani-Cloth AF					
Sani-Cloth AF3					
Sani-Cloth Bleach Wipes	✓	✓	✓	✓	✓
Sani-Cloth HB	✓	✓	✓	✓	✓
Sani-Cloth Plus	✓	✓	✓	✓	✓
Sekusept	✓	✓	✓	✓	✓
Simple Chlor					
Sporicidin					
Staphene	✓	✓	✓	✓	✓
Steranios (2%)	✓	✓	✓	✓	✓
Steranios (20%)			✓		
Super Sani-Cloth					
T-Spray		✓	✓	✓	✓
T-Spray II	✓	✓	✓	✓	✓
Trigene Advance Wipes	✓	✓	✓	✓	✓
Tristel Solo	✓	✓	✓	✓	✓
Tristel Wipes			✓		

Table 5: Disinfectant Compatibility with Retired FUJIFILM SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e	L38 L38e	L52 L52e Vet Only	P10	P17
Virex 256					
Virex II 256					
Virex TB					
Virox 5 Wipes	✓	✓	✓	✓	✓
Virox 5 RTU Wipes					
Wet Task Wipes	✓	✓	✓	✓	✓
Wet Wipe Disinfection	✓	✓	✓	✓	✓
Wex-Cide	✓	✓		✓	✓
Wipes Plus Disinfecting Wipes	✓	✓	✓	✓	✓
Wood Wyant Ultra Wipes					

✓ - Acceptable for use

1 Disinfectant has been tested by FUJIFILM SonoSite for disinfection effectiveness on FUJIFILM SonoSite transducers.

Retired Specialty Transducers

You can disinfect the transducer, scanning surface, and cable using a recommended disinfectant or device.

Clean the transducer prior to disinfecting to remove any particulate matter or bodily fluids. See [“To clean a transducer”](#) on page 26.

The retired specialty transducers listed below correspond to those listed in the table [“Retired Ultrasound Systems and Docks”](#) on page 2.

Table 6: Disinfectant-Device Compatibility with Retired FUJIFILM SonoSite Specialty Transducers

Device	C8 C8e C8x	ICT	SLA	SLT
Trophon EPR*	✓	✓	✓	

✓ - Acceptable for use

* For information and guidance on disinfection effectiveness and appropriate clinical use, see the manufacturer’s user manual.

Table 7: Disinfectant Compatibility with Retired FUJIFILM SonoSite Specialty Transducers

Disinfectant	C8 C8e C8x	ICT	SLA	SLT
Airkem A-33	✓	✓	✓	✓
Alcohol, Ethanol				
Anios Wipes	✓	✓	✓	✓
Aniosyme DD1 (5%)	✓	✓	✓	✓
Anioxyde 1000				✓
Asepti-HB				
Asepti-Wipe II	✓	✓	✓	✓
Bacoban	✓	✓	✓	✓
Baxedin Pre-Op	✓	✓	✓	✓
CaviWipes	✓	✓	✓	✓

Table 7: Disinfectant Compatibility with Retired FUJIFILM SonoSite Specialty Transducers (continued)

Disinfectant	C8 C8e C8x	ICT	SLA	SLT
Cidalkan	✓	✓	✓	✓
Cidalkan Lingettes	✓	✓	✓	✓
Cidex	✓	✓	✓	✓
Cidex OPA	✓	✓	✓	✓
Cidex Plus	✓	✓	✓	✓
Cleanisept Wipe	✓	✓	✓	✓
Clinell Universal Sanitising Wipes		✓	✓	
Clorox Germicidal Wipes	✓	✓	✓	✓
Control III	✓	✓	✓	✓
Coverage Germicidal Surface Wipes	✓	✓	✓	✓
Coverage Plus Germicidal Surface Wipes	✓	✓	✓	
Coverage Spray				
Dispatch Wipes	✓	✓	✓	✓
Dynacide PA	✓	✓	✓	✓
Echo Clean Lingettes		✓	✓	✓
End-Bac II	✓	✓	✓	✓
Endosporine	✓	✓	✓	✓
Envirocide				
Gluco Chlor Wipes	✓	✓	✓	✓
Hydrogen Peroxide (3%)	✓	✓	✓	✓
Incidin Plus (1%)	✓	✓	✓	✓

Table 7: Disinfectant Compatibility with Retired FUJIFILM SonoSite Specialty Transducers (continued)

Disinfectant	C8 C8e C8x	ICT	SLA	SLT
Incidin Plus (3%)	✓	✓	✓	✓
Korsolex Extra	✓	✓	✓	✓
Madacide 1		✓	✓	✓
Medzyme	✓	✓	✓	✓
Metricide OPA Plus	✓	✓	✓	✓
MetriSponge	✓	✓	✓	✓
PerCept RTU Wipes		✓	✓	✓
Rely+On PeraSafe				✓
Sani-Cloth Bleach Wipes	✓	✓	✓	✓
Sani-Cloth HB	✓	✓	✓	✓
Sani-Cloth Plus	✓	✓	✓	✓
Sekusept	✓	✓	✓	✓
Staphene	✓	✓	✓	✓
Steranios (2%)	✓	✓	✓	✓
Steranios (20%)				✓
T-Spray		✓	✓	✓
T-Spray II	✓	✓	✓	✓
Trigene Advance Wipes	✓	✓	✓	✓
Tristel Duo	✓			
Tristel Solo	✓	✓	✓	✓
Tristel Wipes				✓

Table 7: Disinfectant Compatibility with Retired FUJIFILM SonoSite Specialty Transducers (continued)

Disinfectant	C8 C8e C8x	ICT	SLA	SLT
Virex 256				
Virex TB				
Virox 5 Wipes	✓	✓	✓	✓
Wet Task Wipes	✓	✓	✓	✓
Wet Wipe Disinfection	✓	✓	✓	✓
Wex-Cide	✓	✓	✓	
Wipes Plus Disinfecting Wipes		✓	✓	✓

✓ - Acceptable for use

Retired Accessories

You can disinfect the exterior surface of the accessories using a recommended disinfectant.

The retired accessories listed below correspond to those listed in the table “Retired Ultrasound Systems and Docks” on page 2.

Table 8: Disinfectant Compatibility with Retired FUJIFILM SonoSite Accessories

Disinfectant	H-Universal Stand	MDS Lite, MDS Lite II	Basic Stand, Footswitch, iLook Stand, MDSe, MDSm, S Series Stand, ScanPack, SiteCharge, SiteStand	System battery	iLook docking station	PowerPack	PowerPark	ECG Cable	Remote Control	Power Supply	Triple Transducer Connect
Airkem A-33	✓					✓	✓			✓	✓
Alcohol, Ethanol											
Alcohol, Isopropyl (100%)	✓										✓
Alcohol, Isopropyl (70%)		✓	✓	✓	✓			✓			
Anios Wipes	✓					✓	✓		✓	✓	✓
Aniosyme DD1 (5%)	✓					✓	✓			✓	✓
Anioxyde 1000	✓					✓	✓		✓	✓	✓
Asepti-HB	✓					✓	✓			✓	✓
Asepti-Steryl						✓				✓	✓
Asepti-Wipe II	✓					✓	✓			✓	✓
Baxedin Pre-Op						✓					
CaviWipes	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Cidalkan										✓	✓
Cidalkan Wipes											

Table 8: Disinfectant Compatibility with Retired FUJIFILM SonoSite Accessories (continued)

Disinfectant	H-Universal Stand	MDS Lite, MDS Lite II	Basic Stand, Footswitch, iLook Stand, MDSe, MDSm, S Series Stand, ScanPack, SiteCharge, SiteStand	System battery	iLook docking station	PowerPack	PowerPark	ECG Cable	Remote Control	Power Supply	Triple Transducer Connect
Cidex	✓					✓	✓			✓	✓
Cidex OPA	✓					✓	✓			✓	✓
Cidex Plus	✓					✓	✓			✓	✓
Clinell Universal Sanitising Wipes	✓					✓				✓	✓
Clorox Germicidal Wipe						✓				✓	
Control III	✓					✓	✓			✓	✓
Coverage Germicidal Surface Wipes	✓									✓	✓
Coverage Plus Germicidal Surface Wipes										✓	✓
Coverage Plus Wipes									✓		
Coverage Spray	✓					✓				✓	✓
Coverage Wipes						✓			✓		
DisCide Ultra Disinfecting Towelettes						✓				✓	✓
Dynacide PA	✓					✓	✓			✓	✓
Echo Clean Lingettes						✓	✓			✓	✓

Table 8: Disinfectant Compatibility with Retired FUJIFILM SonoSite Accessories (continued)

Disinfectant	H-Universal Stand	MDS Lite, MDS Lite II	Basic Stand, Footswitch, iLook Stand, MDS, MDSm, S Series Stand, ScanPack, SiteCharge, SiteStand	System battery	iLook docking station	PowerPack	PowerPark	EKG Cable	Remote Control	Power Supply	Triple Transducer Connect
End-Bac II						✓				✓	✓
Endosporine	✓					✓	✓		✓	✓	✓
Envirocide	✓					✓	✓			✓	✓
Gluco Chlor Wipes						✓				✓	
Hydrogen Peroxide (3%)	✓					✓	✓		✓ ¹	✓	✓
Incidin Plus (1%)	✓					✓	✓			✓	✓
Incidin Plus (3%)	✓					✓	✓			✓	
Madacide 1						✓				✓	
Medzyme	✓					✓	✓			✓	
Metricide OPA Plus	✓					✓	✓		✓	✓	✓
Mikrobac Forte 2%											
Mikrobac Tissues											
Nu-Cidex							✓				✓
Oxivir TB Wipes											
PerCept RTU Wipes	✓					✓	✓		✓	✓	
Rely+On PeraSafe	✓					✓	✓		✓	✓	✓
Sani-Cloth AF											✓

Table 8: Disinfectant Compatibility with Retired FUJIFILM SonoSite Accessories (continued)

Disinfectant	H-Universal Stand	MDS Lite, MDS Lite II	Basic Stand, Footswitch, iLook Stand, MDSe, MDSm, S Series Stand, ScanPack, SiteCharge, SiteStand	System battery	iLook docking station	PowerPack	PowerPark	ECG Cable	Remote Control	Power Supply	Triple Transducer Connect
Sani-Cloth Bleach Wipes						✓				✓	
Sani-Cloth HB						✓			✓	✓	
Sani-Cloth Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Simple Chlor											
Sporicidin	✓					✓	✓		✓	✓	✓
Staphene						✓				✓	
Steranios 2%	✓					✓	✓			✓	✓
Steranios 20%	✓					✓	✓			✓	✓
Super Sani-Cloth			✓	✓	✓	✓		✓	✓ ¹	✓	✓
T-Spray	✓					✓				✓	
T-Spray II	✓					✓	✓		✓	✓	✓
Trigene Advance Wipes										✓	
Tristel Solo	✓					✓	✓		✓	✓	✓
Tristel Wipes										✓	
Virex 256	✓					✓	✓			✓	
Virex II 256											
Virex TB	✓					✓				✓	

Table 8: Disinfectant Compatibility with Retired FUJIFILM SonoSite Accessories (continued)

Disinfectant	H-Universal Stand	MDS Lite, MDS Lite II	Basic Stand, Footswitch, iLook Stand, MDS, MDSm, S Series Stand, ScanPack, SiteCharge, SiteStand	System battery	iLook docking station	PowerPack	PowerPark	ECG Cable	Remote Control	Power Supply	Triple Transducer Connect
Virox 5 Wipes	✓					✓	✓		✓	✓	✓
Wet Task Wipes	✓					✓				✓	
Wex-Cide	✓					✓	✓		✓	✓	✓
Wipes Plus Disinfecting Wipes	✓					✓				✓	✓
Wood Wyant Ultra Wipes	✓										✓

✓ - Acceptable for use

1 Not for use on the microphone clip. (The microphone clip is detachable from the microphone.)

Cleaning Transducers

Clean the transducer prior to disinfecting to remove any particulate matter or bodily fluids.

To clean a transducer

- 1 Disconnect the transducer from the ultrasound system.
- 2 Remove any transducer sheath.
- 3 Clean the surface using a soft cloth lightly dampened in a mild soap or detergent cleaning solution.
Apply the solution to the cloth rather than the surface.
- 4 Rinse with water or wipe with water-dampened cloth; then wipe with a dry cloth.

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

United States law restricts this device to sale by or on the order of a physician.

iLook, MicroMaxx, TITAN, SonoSite, and the SonoSite logo are registered and unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

Part Number: P06703-10

Publication Date: January 2016

Copyright © 2016 FUJIFILM SonoSite, Inc. All Rights reserved.



P06703-10

