

Voice Assist Microphone Installation Guide

| Manufacturer | EC Authorized Representative | UK Responsible Person | Australia Sponsor |
|--|--|--|--|
| FUJIFILM SonoSite, Inc. 21919 30th Drive SE Bothell, WA 98021 USA T: 1-888-482-9449 or 1-425-951-1200 F: 1-425-951-1201 | FUJIFILM SonoSite B.V. Joop Geesinkweg 140 1114 AB Amsterdam, The Netherlands | Fujifilm UK Limited Fujifilm House Whitbread Way Bedford MK42 0ZE United Kingdom | FUJIFILM SonoSite Australasia Pty Ltd 114 Old Pittwater Road BROOKVALE, NSW, 2100 Australia |

SONOSITE is a registered trademark of FUJIFILM SonoSite, Inc., in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation. All other trademarks are the property of their respective owners.

Patent: www.sonosite.com/patents




P27323-01

Published Jul 31, 2023


Copyright © 2023 FUJIFILM SonoSite, Inc. All rights reserved.

Install the Voice Assist microphone


1. Use the provided screw and hex wrench to attach the mounting bracket to the right side of the stand.

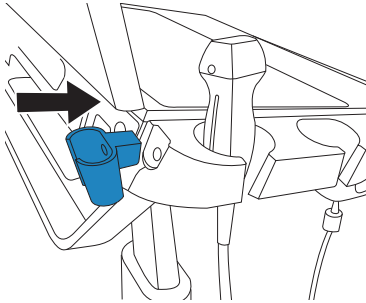

2. Slot the microphone into the mounting fixture.

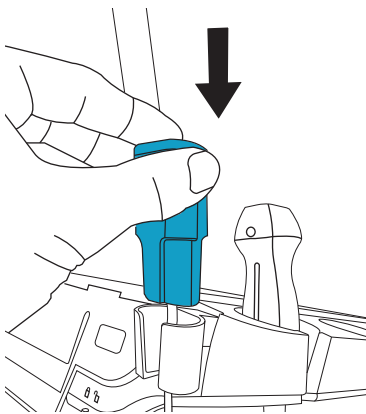

3. Connect the USB cable between the microphone and an available port on the rear of the system. The system should recognize the microphone right away but if not, unplug the USB cable and then plug it back in.

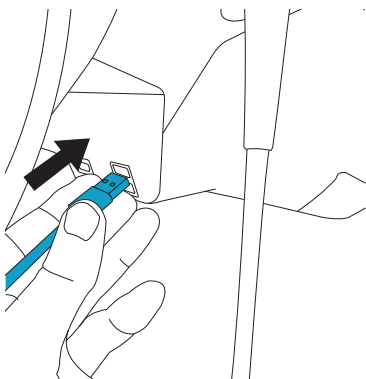

2X

You are now ready to use the Voice Assist feature. See the system user guide for information on the available commands.

FUJIFILM
SONOSITE

P27323-01

