

Notes

- ▶ You must remove any other wireless adapters before the system will use the Wireless and Security Module.
- ▶ This device is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC, the FCC, and Industry Canada.

Cautions


- ▶ This device is restricted to *indoor* use when operated in the 5.15 to 5.25 GHz frequency range.
- ▶ Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment. Operating the wireless USB adapter without a regulatory label or the correct country stamp is illegal. See the documentation from the manufacturer of the wireless USB adapter.
- ▶ Use only the cables that come in the kit to install the wireless USB adapter. Using other USB cables may negatively impact the performance of the adapter.

WARNING


To comply with FCC RF exposure compliance requirements, the antenna used for this transmitter must be installed to provide a separation distance of at least 20 cm from all persons and must not be co-located or operating in conjunction with any other antenna or transmitter.

Unpacking the box


Wireless and Security Module (1)

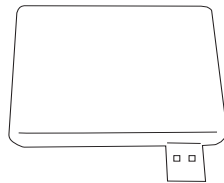

SonoSite SII wireless module holder, front (1)

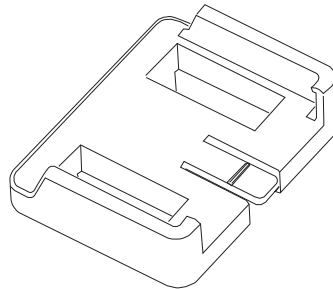


12 inch USB extension cord (1)

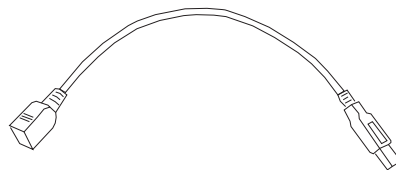

Installing the wireless module


STEP 1

Snap the SonoSite SII wireless module holder onto the transducer holder arm.

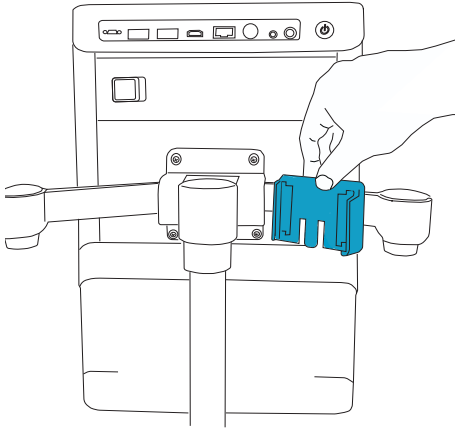


STEP 2

Remove the USB cap, and snap the wireless module into the holder in the orientation shown.

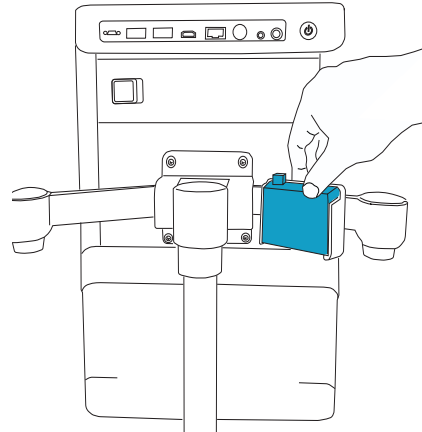

STEP 3

Plug the wireless module into the system with the 12 inch USB cord.

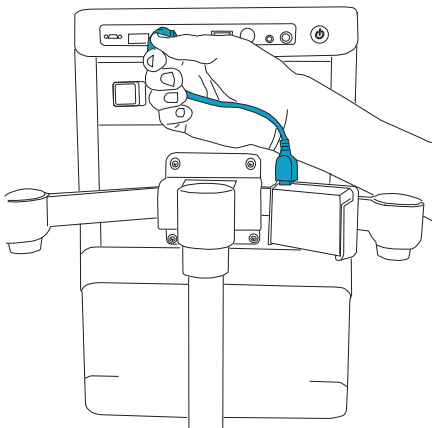

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite
Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia



SonoSite SII and SONOSITE are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Published March 2020

Copyright © 2020 FUJIFILM SonoSite, Inc. All rights reserved.

P28818-01

