SonoSite X-Porte

User Guide Supplement

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or
1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite, X-Porte, and the SonoSite logo are registered and unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

DICOM is a registered trademark of the National Electrical Manufacturers Association.

All other trademarks are the property of their respective owners.

Part number: P23266-01

Publication date: December 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

SonoSite X-Porte User Guide Supplement

Introduction	1
Document conventions	1
Getting help	2
Remote Worksheets	2
Wireless Updates	4
Connectivity settings	4
LVO Setting	5
EFW Calculation	5

Introduction

This user guide supplement provides information on:

- ▶ The Remote Worksheets feature, available with license
- ▶ 5 GHz wireless transmission, available with license
- ▶ LVO setting
- ▶ The Estimated Fetal Weight (EFW) calculation

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the ultrasound system user guide.

Getting help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.

Remote Worksheets

You can now import up to 30 remote worksheets created or edited on a Telexy Qpath E workstation to your SonoSite X-Porte ultrasound system. You can fill in worksheet fields, sign-off, and transfer the remote worksheet data back to the server. You can only import or export Remote Worksheets from or to the Qpath E server.

The following steps should be completed by a network administrator. Refer to the *SonoSite X-Porte User Guide* to set up a local DICOM location and access to the Qpath E server on your system.

To configure access to the remote worksheets server

- 1 Set up a local DICOM location on your ultrasound system.
- 2 Set up the Qpath E server as a DICOM archiver accessible by your system.
- 3 Select **Include Private Tags** to send Remote Worksheet data to the archiver. Remote Worksheet data can only be sent at the end of the exam.

- 4 Set the Qpath E web server as a secure HTTPS address.
 - a Tap **MORE**, and then tap **System Settings**.
 - b Tap **Administration** in the list on the left.
 - c On the **Administration Login** page, enter your administrator user name and password.
 - d Tap **Login**.
 - e Tap **Remote Worksheets**.
 - f The **Remote Worksheets** page appears.
 - g Enter the address of the remote server.
 - h Tap **Verify** to verify the address, or **Clear** to clear the field.
 - i Tap **Back** to return to the **Administration** page.
 - j Tap **Done** to exit.

To import remote worksheets

- 1 On the **Administration Login** settings page, log in as a user with administrative privileges.
- 2 Tap **Remote Worksheets**.

The **Remote Worksheets** page appears.
- 3 Tap **Synchronize** to download the remote worksheets

Note

Successfully synchronizing remote worksheets will completely replace the current set of remote worksheets.

- 4 Tap **OK** once the update is complete. You will return to the **Remote Worksheets** page.
- 5 Tap **Back** to return to the **Administration** page.
- 6 Tap **Done** to exit.

To access and complete a remote worksheet

- 1 Tap **WORKSHEETS**, and then tap **Remote Worksheets** at the bottom of the screen to display the list of remote worksheets
- 2 Select a worksheet from the list, and enter your data.

Note

Active worksheets will display an exclamation point.

3 To sign a worksheet, tap **Not Signed** at the top of the worksheet.

On the **WORKSHEETS** page, a check mark appears indicating that the worksheet has been signed.

Note

To be able to sign a worksheet, the worksheet must be configured to include a signature, and the physician **User** ID must be filled out under **Facility** in the patient information form.

4 Tap **Done**.

Note

You must sign each remote worksheet separately.

Wireless Updates

5 GHz wireless transmission is now available by license in some countries on the system. The SonoSite X-Porte ultrasound system contains an internal IEEE 802.11 transmitter that uses the Industrial, Scientific, and Medical (ISM) frequency bands from 2.400 to 2.4835 GHz, from 5.150 to 5.250, and from 5.725 to 5.850 GHz. The transmitter implements four different methods of transmission:

- ▶ IEEE 802.11a with Orthogonal Frequency Division Multiplexing (OFDM) at 11 dBm \pm 2 dBm @ 54 Mbps
- ▶ IEEE 802.11b with Direct Sequence Spread Spectrum (DSSS) at 16 dBm \pm 2.0 dBm @ 11 Mbps
- ▶ IEEE 802.11g with Orthogonal Frequency Division Multiplexing (OFDM) at 13 dBm \pm 2.0 dBm @ 54 Mbps
- ▶ IEEE 802.11n with Orthogonal Frequency Division Multiplexing (OFDM) at 12 dBm \pm 2.0 dBm (802.11gn) @ MCS7

Connectivity settings

To configure a wireless connection

- 1 Tap **MORE**, and then tap **System Settings**.
- 2 Tap **Connectivity** in the list on the left.
- 3 On the **Connectivity** settings page, tap **DICOM Setup**.
- 4 Tap **Config**, then select **Location**.
- 5 On the DICOM **Location** page, select **New** to configure a new wireless connection.
Default information will automatically populate the fields.
- 6 Edit the information fields, as needed.
- 7 Select the desired settings under **Transfer images** and **JPEG Compression**.
- 8 Select 2.4 GHz, 5 GHz, or both from the **Network** type.
- 9 Select **DHCP** to automatically provide configuration information.

10 Tap **Wireless Configuration**....

11 On the next page, select **New**.

12 Edit the information fields, as needed.

13 Select an **Authentication Type**, **Client Certificate**, **Private Key**, and **Certification Authority** under the drop down menus.

Note

These are available if you have imported wireless certificates to your system.

14 Enter your password.

15 Select **Save**.

16 Select **Done** to exit system settings.

Note

If a wireless connection has already been configured, select it from the wireless drop down list on the DICOM **Location** page.

LVO Setting

The mechanical index (MI) setting has been lowered for the LVO control to a fixed 0.2. Tap **LVO** to lower the MI of the system. LVO is only available if THI is turned on.

EFW Calculation

Your system will calculate Estimated Fetal Weight (EFW) calculations using biparietal diameter (BPD), head circumference (HC), abdominal circumference (AC), and femur length (FL) values that are within range. If the BPD and HC values are out of range, the system calculates the EFW using only the AC and FL values.

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