SonoSite iViz

User Guide Errata

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite, the SonoSite logo, and SonoSite iViz are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P22991-01

Publication date: October 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

SonoSite iViz User Guide Errata

Introduction	1
Document conventions	1
Getting Help	2
Getting Started	2
About SonoSite iViz	2
Configuring SonoSite iViz	3
Configuring EMR settings and preferences	3
Managing Patient Records	3
Accessing patient information	3
Searching for a patient record	3
Managing studies	3
Viewing scheduled studies	3
Sharing a report	4
Troubleshooting and Maintenance	4
DICOM common problems	4
SonoSite iViz DICOM FAQ	4

Introduction

The EMR system integration feature is no longer offered on the iViz ultrasound system. Revisions to the system user guide will be made in the next update.

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the ultrasound system user guide.

Getting Help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.

Getting Started

The following content has been updated.

About SonoSite iViz

SonoSite iViz is a portable, hand-held device that acquires and displays high-resolution, real-time ultrasound images. Features available on the system include:

- ▶ 2D scanning mode with color Doppler
- ▶ Measurement and calculation support
- ▶ Image labeling
- ▶ DICOM support
- ▶ On-board help videos

Configuring SonoSite iViz

The following content has been updated.

Configuring SonoSite iViz settings

SonoSite iViz can be adapted to a wide variety of conditions. For example, from the SonoSite iViz **Settings** screen, you can:

- ▶ Set user preferences, such as units of measure, right or left-handed operation, standard clip length and type.
- ▶ Customize the obstetrics tools the system uses to help you calculate gestational age.
- ▶ Add to or change the standard labels available in various exam types.
- ▶ Set up or change DICOM profiles for each unique location/institution and connect to the local servers.
- ▶ Modify patient search parameters, such as name, ID, date of birth, and procedure.

Configuring EMR settings and preferences

This section has been removed.

Managing Patient Records

The following content has been updated.

The Patient module offers tools for searching and managing patient exam records, which are referred to in the system as *studies*. The Patient module enables you to search the worklist server for specific studies, update patient information, create new studies, and save exams.

Accessing patient information

Searching for a patient record

The Search function displays results only when SonoSite iViz is connected to a Worklist server.

Managing studies

The following content has been updated.

Viewing scheduled studies

The list of scheduled studies displays only when SonoSite iViz is connected to a Worklist server.

Sharing a report

This section has been removed.

Troubleshooting and Maintenance

DICOM common problems

The following content has been updated.

SonoSite iViz DICOM FAQ

- ▶ **How do I know my exam/study was successfully transferred to the DICOM archive?** After you finish transferring the exam or study, tap the **Refresh** button in the top right corner to refresh the screen.
 - ▶ If the transfer was successful, the word **Archived** appears in the **Status** column.
 - ▶ If the transfer is not yet complete, the word **Pending** appears in the **Status** column.
 - ▶ If the transfer has failed, the word **Failed** appears in the **Status** column.
 - ▶ If no transfer has been attempted, the message **Not Archived** appears in the **Status** column.

To get a detailed status, including the destination, date and time, and a list of attempts, tap the status of the exam/study.

- ▶ **When transferring DICOM data to a USB storage device, how do I know when I can remove the device?** As soon as the USB transfer is complete, the message **DICOM studies transferred successfully** appears in the top left part of the screen. You may also see a message that says **Studies transferred successfully**. When either of these messages appear, you can safely remove the USB storage device.
- ▶ **How do I know if my images/clips have been emailed successfully?** The message **DICOM studies transferred successfully** appears in the upper left part of the screen as soon as the email transfer is complete.
- ▶ **How many scheduled worklist procedures can I query and view at one time?** SonoSite iViz can display up to 100 procedures.
- ▶ **Does SonoSite iViz support DICOM Print?** DICOM Print is not supported at this time.
- ▶ **Does SonoSite iViz support DICOM Storage Commitment service?** The DICOM Storage Commitment service is not supported at this time.

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